



F  **FLEX**
STAR



Your Qualified Flex-Star Lab

***Invisible Partial Dentures,
Comfortable & Metal-Free***

 **NOBILIUM™**


Division of CMP Industries LLC
413 North Pearl Street • Albany, New York 12207 • USA
Toll-Free: 800.833.2343 • www.nobilium.com

Aesthetic Comfort

Flex-Star Partials Are Comfortable & Look Natural With NO UNSIGHTLY METAL FRAMEWORK TO SHOW!

Proven Biocompatibility

The US Pharmacopeia* rating for Flex-Star V base polyamide is Class VI, the highest rating possible. The pigments used are food-grade.

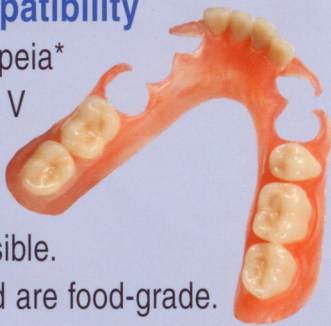

Easy to Adjust

Easier to adjust and polish compared to other leading brands of partial materials.

Familiar Shades



Standard Pink



Light Pink



Maharry



White


*U.S. Pharmacopeia

The United States Pharmacopeia (USP) is the official public standards-setting authority for all prescription and over-the-counter medicines, dietary supplements, and other healthcare products manufactured and sold in the United States. For more information, visit www.usp.org.



Hand-Crafted

Each case is hand-crafted and comes in a sealed bag along with dentist and patient instructions.

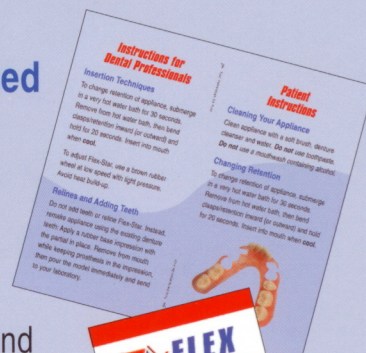

Which Partial will your Patient Prefer?

Conventional



Flex-Star





F  **FLEX**
STAR

***Aesthetic
Comfort***



 **NOBILIUM™**

Manufactured by CMP Industries LLC

413 North Pearl Street • Albany, New York 12207 • USA

Toll-Free: 800.833-2343 • www.nobilium.com

***Instructions for Dental
Professionals and Patients***

Instructions for Dental Professionals

Insertion Techniques

To change retention of appliance, submerge in a very hot water bath for 30 seconds. Remove from hot water bath, then bend clasps/retention inward (or outward) and hold for 20 seconds. Insert into mouth when **cool**.

To adjust Flex-Star, use a brown rubber wheel at low speed with light pressure. Avoid heat build-up.

Relines and Adding Teeth

Do not add teeth or reline Flex-Star. Instead, remake appliance using the existing denture teeth: Apply a rubber base impression with the partial in place. Remove from mouth while keeping prosthesis in the impression, then pour the model immediately and send to your laboratory.

Patient Instructions

Cleaning Your Appliance

Clean appliance with a soft brush, denture cleanser and water. **Do not use toothpaste.**
Do not use a mouthwash containing alcohol.

Changing Retention

To change retention of appliance, submerge in a very hot water bath for 30 seconds. Remove from hot water bath, then bend clasps/retention inward (or outward) and hold for 20 seconds. Insert into mouth when **cool**.

